A GUIDE TO  
*simplify your life*  
WITH THE FUN CUP

**FUN**  
FACTORY

## Wash me!

Boil your FUN CUP before first use and wash it with mild soap and warm water after emptying.

Wash your hands before and after each use.

Between cycles, boil it, let it dry and store it in its bag.

The use of cleaning agents containing chlorine can lead to changes in the color and feel of the product.

To clean the bag, wash it with soap and water, then let it air dry.

## Remove and empty me!

Use your pelvic floor muscles to push down on the cup, then pinch it between two fingers to break the seal, or insert a finger and break the seal manually by pressing against the edge of the cup. Once you've broken the seal, pinch the cup and pull it out, keeping it upright.

Then, empty it into the toilet. Take it out, empty and clean regularly.

## Insert me!

If it's your first time using a menstrual cup, try the FUN CUP when you have a little time to experiment. Find a comfortable position with your legs apart – sit on the toilet or stand with one leg propped up.

Fold the cup (see diagrams). Keeping it folded, insert the FUN CUP into your vagina. A little water-based lube can help.

Top: C-fold, bottom: punch-fold

Push it in as far as it will go, then let go. Making a scooping motion down and then toward the back while you insert can help position the cup well. The cup will open, covering your cervix.

To create a seal, twist the cup (either holding the sides or the firm silicone tip) while bearing down with your muscles. Make sure the cup is covering your cervix by swirling a finger around the sides. Give it a gentle tug to make sure it's created a seal. If it has, you should be able to feel a little resistance.

## English

FUN CUP is a menstrual cup made of medical silicone and is only suitable for vaginal use. Read the instructions for use before use and learn about the risks associated with using menstrual products. Wash your hands thoroughly with soap before inserting or removing the FUN CUP. Boil the FUN CUP in water for 3 minutes before use and between uses.

### **INSERT:**

- Before inserting it, make sure that the FUN CUP is sterile and defect-free.
- Fold the FUN CUP – see pictures.
- Insert it into your vagina.
- Push it until you have overcome a natural resistance and release the handle.

The FUN CUP will then open and be hermetically enclosed by the walls of your vagina. To make sure, slide your finger gently around the top opening. If you need to help it, grasp the cup at the top and turn it slightly. Then when you pull slightly on the top and feel resistance, the vacuum is working.

### **REMOVE:**

- Remove the FUN CUP every 6 hours (recommended) and after a maximum of 12 hours, more often if you have heavy periods
- Use your pelvic floor muscles to push the cup down, then pinch it together with two fingers and release the vacuum. Or press a finger against the top of the FUN CUP to release the vacuum.

- Then carefully pull the cup out with the opening facing upwards and empty it into the toilet.

### **CLEAN:**

- Clean the Fun CUP with water and a little mild soap and then boil it in pure boiling water for 3 minutes to sterilize it. Pay particular attention to the hard-to-reach holes when cleaning. Dry it off.
- Clean the bag with water and a little mild soap and let it air dry.
- Do not use cleaning agents other than those specified.
- Store the products in a dry place that is protected from direct sunlight.

### **PLEASE BEAR IN MIND:**

TSS (Toxic Shock Syndrome) is a very rare condition that can occur through the use of menstrual products and can be fatal without treatment. It is very unlikely to occur if you use and clean the FUN CUP correctly. Symptoms of TSS include sudden fever, chills, nausea, vomiting, muscle pain, drop in blood pressure, dizziness and drowsiness. If the symptoms appear, remove the FUN CUP and see a doctor. If TSS has already occurred or if there are injuries and infections in the genital area, the use of the FUN CUP and other inserted menstrual products is not recommended.

We also recommend consulting your doctor before using a menstrual product for the first time. More information is available at [www.thefuncup.com](http://www.thefuncup.com)

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**FUN FACTORY** GmbH